


Points to Practice

BD PosiFlush™ (NaCl 0.9%) Pre-Filled Flush Syringe




For surgical ANTT requiring critical management of an aseptic field use BD PosiFlush™ XS

1. Check patient's identity
2. Check packaging is intact then open
3. Check syringe contents and verify expiry date
4. Click to unstick. Gently depress the plunger until you feel/hear a click. This will free the plunger
5. Unscrew tip cap to expel air
6. Clean catheter access port according to Trust policy

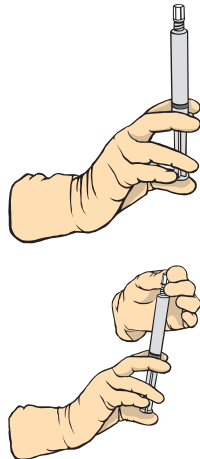

For standard ANTT use BD PosiFlush™ SP

7. Remove cap then attach syringe to the catheter
8. Use push/pause technique to administer, which creates turbulent flow
9. Finish flush using positive pressure. If using an extension with a clamp, clamp line while thumb remains on syringe plunger
10. Dispose into clinical waste or sharps bin according to Trust policy

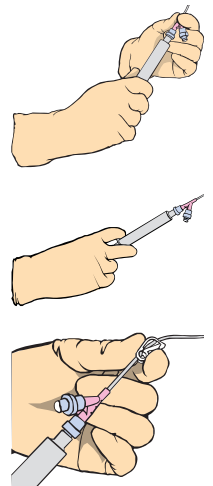

NOT FOR RECONSTITUTION OF DRUGS OR PATIENT REHYDRATION



(Refer to Instructions For Use for more information)
Recommended by ANTT to promote safer aseptic practice

BD Switzerland SàRL, Terre Bonne Park - A4, Route de Crassier 17, 1262 Eysins, Switzerland
Tel: +41 21 556 30 00

bd.com

